

Your satisfaction is important to us, please let us help. We are providing you this handbook and **User Instructions** for the most requested products. Additional User Instructions will be included with the product.

If you have any questions or concerns regarding the product, our support representatives are available to assist you. 801-800-8375.

If you have damaged or missing parts, or if the product is the wrong size, a replacement part, part or another size will be shipped free of charge, directly to you. Please call: 801-800-8375.

Caution: You should never use damaged parts! Instructions for proper installation and use are included with each product.



*LaceIT LSO Video



*Posterior Panel Video



**DryTex Knee Video 1



**DryTex Knee Video 2



**DryTex Knee Video 3



***djOrtho Shoulder Video

*LaceIT LSO (Back Brace) Instruction Video:

<https://www.youtube.com/watch?v=PKQXa8bRf-c>

*LaceIT Posterior Panel Instruction Video

<https://www.youtube.com/watch?v=SCACmRhkaG8>

****DjOrtho DryTex Hinge Knee Brace Instruction Videos:**

Video 1 demonstrates how to wrap the DryTex Hinge Knee Brace around your knee.

<https://youtu.be/gBTWbGJJffc>

Video 2 demonstrates two types of DryTex Hinge Knee braces available from DjOrtho.

<http://www.youtube.com/watch?v=8YMXrPoa3ss>

Video 3 illustrates how to use the components that come with the DryTex Hinge Knee Brace.

<http://www.youtube.com/watch?v=7Cs2XzJhGvo>

*****DjOrtho Shoulder Cradle Instruction Video:**

http://youtu.be/8TS_jhKKFIE

LaceIT X640 L.S.O. Fitting Instructions

1. Place handles as close to the closing system as possible.
2. Be sure handles are next to closing system. Stand holding product side panels parallel to floor. Pull outward to fully extend cord.
3. Place closing system around body, making sure closing system is centered on spine. Supports holding closing system must be equally spaced on each side of spine. Adjust up or down spine to determine best support location.
4. Once location is determined, pull slightly outward on side panels to maximize open position before using fabric fasteners to close front, right over left.
5. Pull handles away from fabric fastener and across body, placing handles anywhere on fabric fastener strip encircling side panels once desired support is achieved. Loosen the same way.

User Tips

Pull handles only slightly at first until you get used to the power.

Tighten to stand, loosen to sit. You control the tension.

Pulling in stomach muscles before tightening makes pulling handles easier.

Do not wear tight for long periods of time unless instructed by physician.

The product was designed to enable adjustment throughout the day to proceed more-or-less support as needed.

To disconnect closing system, simply pull closing system away from fabric fastener. Attach closing system by placing closing system on fabric fastener.

Removal of Anterior Panel to the LaceIT X640

1. Locate the inside lining of the left side panel.
2. Locate the anterior panel pocket.
3. Line up the anterior panel pocket, and locate and remove the rigid plastic insert.

Attaching the Solid Posterior Frame

1. Lift-up the elastic straps located on the inside, posterior portion of the L.S.O.
2. Place the L.S.O. around the outside portion of the Solid Posterior Frame, making sure the L.S.O. superior edge is facing up.
3. Align the L.S.O. elastic straps over the Solid Posterior Frame alignment tabs, and secure the L.S.O. by sliding the elastic straps through the Solid Posterior Frame tabs.

For more help, visit: www.laceit.com